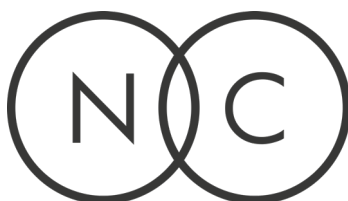
Dahlia Recliner

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EN:

This manual is a simplified guide for using, maintaining, and troubleshooting your electric recliner or lift chair. Please read carefully before use.

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EN:

1. Carefully unpack the chair and remove all plastic.
2. Place the chair on a flat surface.
3. Connect the power cable to a wall outlet (only grounded outlet).
4. Ensure cables are not pinched or damaged.
5. Test the functions before use.
6. Ensure the wheels are retracted. The chair will not function if the wheels are down.
(transport mode deactivated)
7. Do not use during water exposure.

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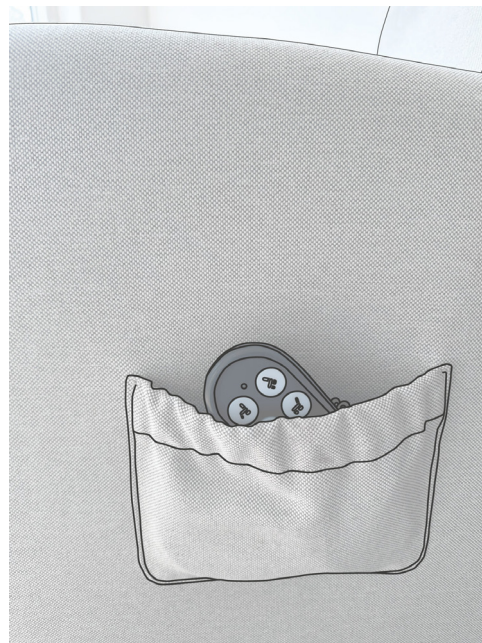
EN:

1. Lift and recliner functions with smooth adjustment.
2. Quiet motor.
3. Simple remote control with up/down buttons.
4. Equipped with a trolley for easy movement.
5. Convenient storage pocket for the remote control.

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EN

The upholstery is removable for easier cleaning. Seat, back and armrest cover.

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EN:

Charge the unit according to the instructions (see cap 2) Monitor during charging and unplug the charger once fully charged. This extends battery life and prevents damage.

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EN:

Clean with a lightly damp cloth (OBS not the mekanism).
Avoid strong chemicals.
Regularly inspect cables and hand control.

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EN:

Problem	Solution
No response	Check power cord, Ensure the wheels are retracted
Motor sounds but does not move	Check for mechanical obstructions
Remote control does not work	Replace battery or check connection

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EN:

By pressing the button, the footrest moves upwards.

By continuing to press the button, the chair moves from the Relax position to the Lift position.

By pressing, the footrest moves upwards; releasing the key, the footrest stops.

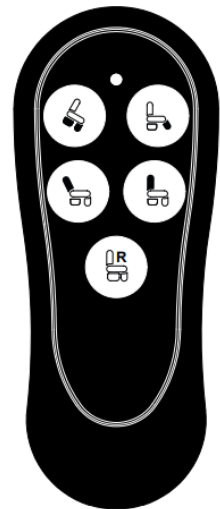
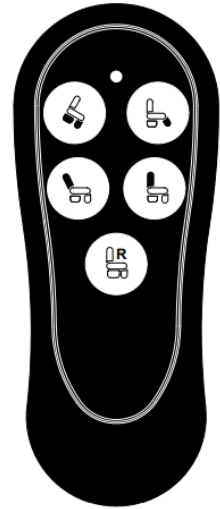
By pressing, the backrest moves backward; releasing the key, the backrest stops.

By pressing the key, the backrest moves upwards and the footrest moves backwards.

By pressing the button, the chair returns to the starting position (home position).

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EN:

- Remove all packaging before use.
- This product is for indoor use in a dry environment only.
- Do not use under the influence of alcohol or medication.
- Only suitable for persons with full physical and mental capability, unless supervised.
- Keep children and pets away when operating. Children must not play with the chair.
- Moving parts may cause injury – keep body parts, children and objects clear.
- Do not sit on the legrest, armrests or backrest – may tip over or damage the chair.
- Use only on a flat, stable surface.
- Never exceed the maximum load as specified by the manufacturer.
- Ensure nothing is underneath the chair before operating.
- Unplug the chair when not in use or when cleaning.
- Never move the chair while plugged into a power outlet.
- Ensure the power cable is not a tripping hazard and not stored under the chair.
- Test all functions before first use.
- The chair is not fitted with battery backup unless specified at purchase.
- Do not use for medical support purposes.
- Do not leave in raised position when not in use.
- Only one person may use the chair at a time.
- Do not use with users showing violent behavior.
- NOTE! The chair is not intended for transporting people.