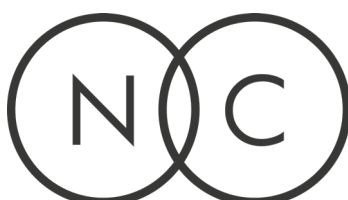
## Dahlia Recliner

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EN

This manual is a simplified guide for using, maintaining, and troubleshooting your electric recliner or lift chair. Please read carefully before use.

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EN

1. Carefully unpack the chair and remove all plastic.
2. Place the chair on a flat surface.
3. Connect the power cable to a wall outlet (only grounded outlet).
4. Ensure cables are not pinched or damaged.
5. Test the functions before use.
6. Ensure the wheels are retracted. The chair will not function if the wheels are down.  
(transport mode deactivated)
7. Do not use during water exposure.

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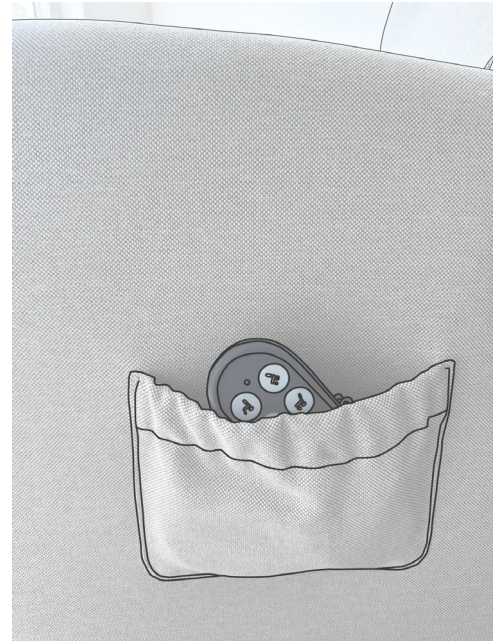
EN

1. Lift and recliner functions with smooth adjustment.
2. Quiet motor.
3. Simple remote control with up/down buttons.
4. Equipped with a trolley for easy movement.
5. Convenient storage pocket for the remote control.

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EN

The upholstery is removable for easier cleaning. Seat, back and armrest cover.

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EN

Charge the unit according to the instructions (see cap 2) Monitor during charging and unplug the charger once fully charged. This extends battery life and prevents damage.

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EN

Clean with a lightly damp cloth (OBS not the mekanism).  
Avoid strong chemicals.  
Regularly inspect cables and hand control.

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EN

Problem	Solution
No response	Check power cord, Ensure the wheels are retracted
Motor sounds but does not move	Check for mechanical obstructions
Remote control does not work	Replace battery or check connection

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EN

By pressing the button, the footrest moves upwards.

By continuing to press the button, the chair moves from the Relax position to the Lift position.

By pressing, the footrest moves upwards; releasing the key, the footrest stops.

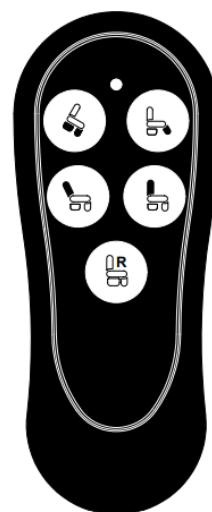
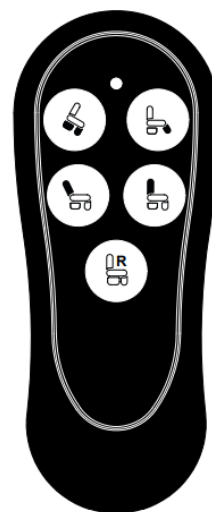
By pressing, the backrest moves backward; releasing the key, the backrest stops.

By pressing the key, the backrest moves upwards and the footrest moves backwards.

By pressing the button, the chair returns to the starting position (home position).

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EN

- This product is intended for indoor use in a dry environment only.
- Do not use the chair if exposed to water or moisture.
- Unplug the chair when not in use or during cleaning.
- Never move the chair while connected to a power outlet.
- Ensure the power cable is not a tripping hazard or placed under the chair.
- Regularly check that cables and connectors are not damaged.
- Keep children and pets away during operation.
- Moving parts may cause injury – keep hands, feet and objects clear of the mechanism.
- Charge the battery only with the supplied adapter and never leave it charging unattended.
- **NOTE:** The chair is not intended for transporting people.

### **Fire Safety for Lithium-Ion Battery:**

The product contains a lithium-ion battery (26 V, 2.2 Ah).

### **If the battery becomes hot:**

- Stop charging immediately if the battery becomes very hot. Watch for increased smoke and heat. The smoke is toxic, and a fire can spread quickly.
- The product must then be inspected and approved by qualified electrical personnel before charging again.

### **In case of fire:**

- If the product is charging when smoke or fire appears – stop charging by unplugging the cord (without risking your safety).
- Evacuate the area and close the door to the room with the burning item.
- Call emergency services (112).

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### EN

- Remove all packaging before use.
- This product is intended for users with full physical and mental capability unless supervised.
- Do not use the chair under the influence of alcohol or medication.
- Do not sit on the legrest, armrests, or backrest – may cause tipping or damage.
- Use only on a flat and stable surface.
- Never exceed the maximum load as specified by the manufacturer.
- Ensure nothing is underneath the chair before operating.
- Do not leave the chair in a raised position when not in use.
- Only one person may use the chair at a time.
- The chair is not equipped with battery backup unless specified at purchase.

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### EN

- This product contains electronic components and a rechargeable battery and must not be disposed of with household waste.
- The product is subject to WEEE requirements (Waste Electrical and Electronic Equipment) and must be recycled according to the national waste regulations of the country where the product is placed on the market.
- Dispose of the product at an approved recycling facility or electronic waste collection point.
- The battery must be disposed of separately if required by local regulations, the battery is detached by loosening the screws securing it to the mechanism.
- Damaged or end-of-life batteries must be treated as hazardous waste and returned to an authorized recycling point.